

KC02 DORSAL-LUMBAR CORSET(BIVALVE)



INSTRUCTION FOR USE

1. Take the patient's size, tie the straps up and use a pencil to mark the cutting line.
2. Use the polyurethane can indicated for each size.
3. Protect prominent bones with **okmfleecyweb**.
4. Cover the affected part of the body with gauze dressing for protection **okmtub**.
5. It is advisable to do the back part first and, once hardened, the front part.
6. Pour the contents of can No.1 (grey label-isocyanate) into can No.2 (blue label-polyol).
7. Mix the polyurethane for 18-25 (look at the mixing times) seconds and wait until the mixer comes to a complete stop.
8. Pour the polyurethane inside the cloth (back part). extend polyurethane with a roller.
9. After 2-minutes place the prepared orthosis on the patient, making sure that it is in the correct position.
10. Zip the zips up, tie the straps up and hold firmly for 4 minutes to prevent points of pressure.
11. Place elastic bandage on without exerting excessive pressure.
12. After about 7 minutes cut to correct length.
13. Allow the orthosis to harden on the patient for about 20 minutes before removing.
14. When making the front part, try to keep the position of the back part in place and then repeat the same process.
15. Staple the ends of the zip with the okm stapler supplied and zip it up.
16. The polyurethane may shrink slightly during the first hour, so it is important to check this by fitting it on the patient again.
17. The use of Velcro is advisable.

IMPORTANT

- Protect your eyes and hands.
- Handle and open the cans with care.



POLYURETHANE INDICATED FOR EACH SIZE

SIZE	XXS	XS	S	M	L	XL	XXL
PU	KR500 KR500	KR500 KR700	KR700 KR700	KR700 KR900	KR900 KR900	KR900 KR1100	KR1100 KR1100

MIXING TIME

GRAMMES	170	300	500	700	900	1100	1300
RIGID	10"-12"	12"-14"	16"-18"	20"-22"	23"-25"	25"-27"	
SOFT		9"-11"	13"-15"	17"-19"	20"-22"	22"-24"	24"-26"

TIMING > TOTAL 24 MINUTES

