


KC04 LUMBAR CORSET WITH VELCRO MIXED

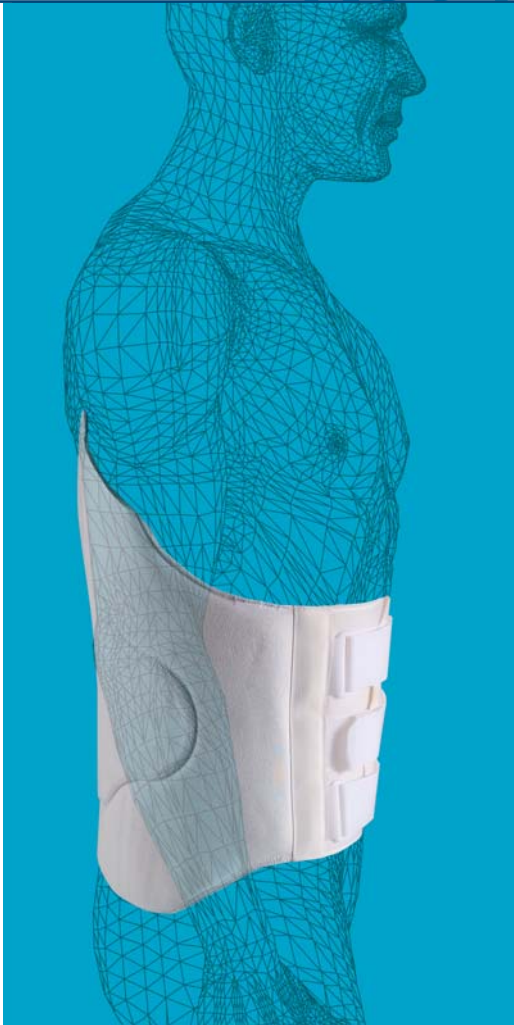

INSTRUCTION FOR USE

1. Take the patient's size, tie the straps up and use a pencil to mark the cutting line. Remove cloth leaving the straps tied on the one side.
2. Use the polyurethane can indicated for each size.
3. Protect prominent bones **okmfleecyweb**.
4. Cover the affected part of the body with gauze dressing for protection **okm tub**.
5. Pour the contents of can No.1 (grey label-isocyanate) into can No.2 (blue label-polyol).
6. Mix the polyurethane for 12-18 (look at the mixing times), seconds and wait until the mixer comes to a complete stop.
7. Pour the polyurethane inside the cloth (the same quantity in both parts). Turn the inner part of the cloth outwards and extend polyurethane with a roller.
8. After 2-2.30 minutes place the prepared orthosis on the patient, fitting the side with the straps tied first and then, tie up on the other side. Close the central Velcro strap without tightening it too much, making sure it is in the correct position.
9. Stretch the cloth by pulling the ends of the cloth and hold firmly for 5 minutes more to prevent points of pressure.
10. Place elastic bandage on without exerting excessive pressure.
11. After about 8 seconds cut to correct length.
12. Close the Velcro
13. Allow the orthosis to harden on the patient for about 20 minutes before removing.
14. The polyurethane may shrink slightly during the first hour, so it is important to check this by fitting it on the patient again.

IMPORTANT

- Protect your eyes.
- Handle and open the cans with care.



POLYURETHANE INDICATED FOR EACH SIZE

SIZE	XXS	XS	S	M	L	XL	XXL
PU	KR170 KRS500	KR170 KRS500	KR300 KRS700	KR300 KRS700	KR300 KRS900	KR300 KRS900	KR500 KRS1100

MIXING TIME

GRAMMES	170	300	500	700	900	1100	1300
RIGID	10"-12"	12"-14"	16"-18"	20"-22"	23"-25"	25"-27"	
SOFT		9"-11"	13"-15"	17"-19"	20"-22"	22"-24"	24"-26"

TIMING > TOTAL 24 MINUTES

