

KC06 MINERVA WITH TRUNK



INSTRUCTION FOR USE

1. Take the patient's size, zip the zips up and use a pencil to mark the cutting line.
2. Use the polyurethane can indicated for each size.
3. Protect prominent bones (especially jaw, glottis and collarbone) with **okmfleecyweb**.
4. Cover the affected part of the body with gauze dressing for protection **okmtub**.
5. It is advisable to do the front part first and, once hardened, the back part.
6. Pour the contents of can No.1 (grey label-isocyanate) into can No.2 (blue label-polyol).
7. Mix the polyurethane for 18-23 (look at the mixing times), seconds and wait until the mixer comes to a complete stop.
8. Pour the polyurethane inside the cloth (front part).
9. Turn the inner part of the cloth outwards and extend polyurethane with a roller.
10. Between 2-2.30 minutes place the prepared orthosis on the patient, making sure that it is in the correct position.
11. Zip the zips up, tie the straps up and hold firmly for 4 minutes more to prevent points of pressure.
12. After about 7 minutes cut to correct length.
13. Staple the ends of the zip with the okm stapler supplied and zip up.
14. Allow the orthosis to harden on the patient for about 20 minutes before removing.
15. When making the back part, try to keep the position of the front part in place and then repeat the same process.
16. The polyurethane may shrink slightly during the first hour, so it is important to check this by fitting it on the patient again.

IMPORTANT

- Protect your eyes and hands.
- Handle and open the cans with care.



POLYURETHANE INDICATED FOR EACH SIZE

SIZE	XXS	XS	S	M	L	XL	XXL
PU	KR500 KR500	KR500 KR500	KR500 KR500	KR700 KR700	KR900 KR900	KR900 KR900	KR900 KR900

MIXING TIME

GRAMMES	170	300	500	700	900	1100	1300
RIGID	10"-12"	12"-14"	16"-18"	20"-22"	23"-25"	25"-27"	
SOFT		9"-11"	13"-15"	17"-19"	20"-22"	22"-24"	24"-26"

TIMING > TOTAL 24 MINUTES

